

**BIO-RAD**

**12/16 DISKS DISPENSER**

50295

---

---

## I - USE

### 1) Square 120 mm x 120 mm Petri dishes

Dispensing of 16 disks (between-disks distance is 3 cm as recommended by the WHO).

Up to four cartridges can be disabled to allow dispensing of only 12 to 15 disks.

### 2) Round 150 mm Petri dishes

Dispensing of 12 disks : the four corner cartridges are disabled to make the dispenser appropriate for dispensing 12 discs into a round 150 mm Petri dish.

### BLOCKING PROCEDURE :

The dispenser being empty, turn it upside down and press as to dispense the disks. The piston to be blocked must be pushed-in with a finger **while simultaneously** rotating (by about one quarter-turn) the appropriate blocking button on the upper side of the dispenser.

Perform this operation for each piston to be blocked.

To release the pistons, it suffices to rotate the four blocking buttons situated on the upper side of the dispenser.

**Note :** Only BIO-RAD disk cartridges can be used with this dispenser.

*Fabriqué et distribué en FRANCE par*

**BIO-RAD**

*3, Boulevard Raymond Poincaré  
92430 MARNES LA COQUETTE*

*Tél. : (33) 01 47 95 60 00*

*Fax.: (33) 01 47 41 91 33*

*Imported in JAPAN by*

**NIPPON BIO-RAD Laboratories**

**BIO-RAD FUJIREBIO, Inc**

*7-18, Higashi-Nippori 5-chome,  
Arakawa-ku*

*TOKYO 116-0014*

05/2000  
code: 863039